# JW Liquid Dam

## JW-Liquid Dam

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### Instructions for use

JW-Liquid Dam is a light-curing gingiva guard which is used similar to a rubber dam for isolating the gingiva from the teeth requiring treatment.

**Indications:**  
For covering the gingiva during bleaching in the dental practice.

**Contraindications:**  
Intolerance to the constituents.

**Preparation:**  
Take appropriate measures (e.g. use cheek and lip retractors) to ensure that the tongue and lips are kept in a safe position during treatment (and the subsequent bleaching procedure) and do not contact the material. Remove the seal of the syringe and replace it with the syringe application tip. Regulate the flow of the material.

**Application:**  
Dry the gingiva. Apply JW-Liquid Dam to a width of approx. 4-6 mm and a thickness of 2 mm along the area to be protected and press the material into the interdental spaces. (Foto 1) Cure for 20 to 40 sec. using a standard light-curing hand lamp. Do not hold the lamp too long on one area. After curing, remove any excess material on the teeth using a scalpel. The bleaching gel can now be applied. (Foto 2)

**Finishing:**  
After finishing, remove the JW-Liquid Dam by lifting the material at one end with an instrument, such as a probe, and pull it off. Remove the syringe application tip and replace it with the seal. (Foto 3)

**Shelf life and storage:**  
JW-Liquid Dam should not be used after the expiry date. Store JW-Liquid Dam in a dark place. Avoid heat and direct sunlight. Store at 3-25 °C.

**Hazard statement:**  
Contains tetramethylene dimethacrylate. May cause an allergic skin reaction.

**Precautionary statements:**  
Wear protective gloves/protective clothing/eye protection/face protection. If skin irritation or rash occurs: Get medical advice/attention.

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Heydent GmbH - Viktor Frankl Str. 20, 86916 Kaufering, Germany
CE GDF GmbH, Dieselstrasse 5-6